

## Application Instructions

### Product: 10" Lumbar Sacral Support with Double Pull Tension Strap

#### Models: 50X, 50X-B

- Form-fitting 10" all-elastic with double pull side panels for additional support.
- High quality hook and loop closure for durability.
- Four flexible stays in back panel for excellent support.



Order # White	Order # Black	Size	Waist Measurement
501	501-B	X-Small	20" - 24"
503	503-B	Small	24" - 30"
505	505-B	Medium	30" - 34"
507	507-B	Large	34" - 38"
508	508-B	X-Large	38" - 42"
509	509-B	2X-Large	42" - 46"
510	510-B	3X-Large	46" - 50"
511	511-B	4X-Large	50" - 54"
512	512-B	5X-Large	54" - 58"

**SIZING:** Measure around waist at widest point.

**HOW TO APPLY:** Apply in a standing position. Unfasten all closures. Place the back of the support around back and upper buttocks area. Pull the sides of the support around until it holds the upper buttocks snugly, secure the hook and loop closure on the support base. Pull the double pull tension straps at the same time on both sides until snug and attach hook and loop closures. Support should fit snug but not bind the groin area when sitting. Readjust if necessary.

**CARE INSTRUCTIONS:** Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:** Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** A4466



326 Main Street  
Harleysville, PA 19438

[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)

©2010 Advanced Orthopaedics, Inc.