

Application Instructions

Product: Advanced Patella Knee Strap

Model: 610

- Indicated for Chondromalacia, Osgood-Schlatter Disease, Patellar Instability, Patellar Tendonitis
- Strap functions dynamically as knee flexes and extends.
- Easy to apply and adjust.
- Universal design fits a range of sizes.



Order

610

Size

Universal

HOW TO APPLY:

Stand or sit with leg straight. Undo one side of strap. Place the front portion of the strap just below the kneecap at the natural indentation. Pull tight and fasten the hook and loop (Velcro-like) closure on the side. Loosen slightly if too tight. Ensure support is not too tight.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com