

Application Instructions

Product: Air-Gel Tennis Elbow System

Model: 2301

- Air-Gel tennis elbow system has three uniquely designed features.
- Air-Gel therapy pad helps reduce swelling.
- Magnetic therapy pad improves circulation.
- Foam tension pad to maximize compression and dissipate force.
- Fits either left or right forearm.



Order #	Size
2301	Universal

HOW TO APPLY:

Open the strap and slide over the forearm so it is positioned just below the elbow with the air-gel pad centered on the tendon where compression is needed. Close the tension strap so it fits comfortably without restricting circulation. Adjust as needed.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A9270



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com