

Application Instructions

Product: Air Lite Arm Sling

Model: 223X

- Breathable mesh material is especially suitable for patients wearing casts.
- Adjustable shoulder strap.
- Thumb loop allows patient to relax hand.
- Hook and loop closure.



Order #	Size
2231	X-Small
2233	Small
2235	Medium
2237	Large
2238	X-Large

HOW TO APPLY:

Unfasten hook and loop closure on strap. Insert arm into mesh pouch with elbow in the pouch corner. Bring strap across back and over the opposite shoulder. Run strap through plastic round loop and adjust to desired length. Press hook and loop closure against strap to secure. Adjust as needed.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4565



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com