

Application Instructions

Product: AIRPRENE Hinged Knee Brace

Model: 90X-AP

- A unique perforated neoprene-like material that retains heat, but not water.
- The perspiration generated by exercise flows through the AIRPRENE to keep the skin dry and comfortable.
- AIRPRENE eliminates excessive heat build-up associated with neoprene.
- Medial and lateral hinge stays provide excellent support.
- Enclosed tubular donut buttress.
- Hyperextension stops at 170°.
- Easy removable hinges for custom fitting.



Order #	Size	Knee Circumference
903-AP	Small	12½" - 14½"
905-AP	Medium	14" - 16"
907-AP	Large	15½" - 18"
908-AP	X-Large	17½" - 20"
909-AP	2X-Large	19½" - 22"

SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY: Unfasten all straps and closures. Insert foot into knee support and slide up over the knee. Align the medial/lateral hinge axis with the center of the knee joint. Support should fit snug but not so tight as to restrict circulation. Hinges should be aligned with the prominent bones on each side of the knee. Secure straps beginning with the top strap. Feed straps through rings and secure.

CARE INSTRUCTIONS: Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L1810 (PDAC Reviewed)



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com