

# Application Instructions

## Product: Body Care Back Support

### Model: 58X

- Double overlapping belt provides additional compression in the abdominal area.
- Double-ply textile back with soft flannel lining.
- Shaped steel stays for a comfortable fit.



Order #	Size	Waist Measurement
583	Small	26" - 30"
585	Medium	30" - 34"
587	Large	34" - 38"
588	X-Large	38" - 42"
589	2X-Large	42" - 46"
589-9	3X-Large	46" - 50"

**SIZING:** Measure around waist at widest point.

### HOW TO APPLY:

Apply in a standing position. Unfasten all closures. Place the back of the support around back and upper buttocks area. Pull the sides of the support around until it holds the upper buttocks snugly, secure the hook and loop closure on the support base. Pull the overlapping abdominal strap until snug and attach hook and loop closures. Support should fit snug but not bind the groin area when sitting. Readjust if necessary.

### CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

### CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** A4466



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)