

# Application Instructions

## Product: Breath Lite Back Support

### Models: 57X, 57X-B

- Designed with a unique ventilated elastic construction making it cool and breathable.
- Provides support and compression to the lower lumbar region and abdominal area.
- Tapered front end, 8" high back panel, which is ideal for shorter or geriatric patients.
- Available in white and black.



| Order # | Order # | Size     | Waist Measurement |
|---------|---------|----------|-------------------|
| White   | Black   |          |                   |
| 573     | 573-B   | Small    | 25" - 29"         |
| 575     | 575-B   | Medium   | 28" - 32"         |
| 577     | 577-B   | Large    | 31" - 35"         |
| 578     | 578-B   | X-Large  | 34" - 38"         |
| 579     | 579-B   | 2X-Large | 37" - 41"         |
| 580     | 580-B   | 3X-Large | 40" - 44"         |

**SIZING:** Measure around waist at widest point.

### HOW TO APPLY:

Apply in a standing position. Unfasten all closures. Place the back of the support around back and upper buttocks area. Pull the sides of the support around until it holds the upper buttocks snugly, secure the hook and loop closure on the support base. Support should fit snug but not bind the groin area when sitting. Readjust if necessary.

### CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:** Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** A4466



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)

©2010 Advanced Orthopaedics, Inc.