

Application Instructions

Product: Canvas Lace-Up Ankle Brace Model: 46X

- Ideal for after cast removal or for moderate sprains.
- Lace - up closure allows for an intimate medial and lateral fit.
- Figure 8 strapping configuration provides extra stability.
- Fits either left or right foot.



Order #	Size	Men's Shoe Size	Women's Shoe Size
461	X-Small	6 – 7	7 – 8
463	Small	7 – 9	8 – 10
465	Medium	9 – 11	10 – 12
467	Large	11 – 13	12 – 14
468	X-Large	13 – 14	14 – 15

SIZING: Size by shoe size. Fits right or left ankle.

HOW TO APPLY:

Wear brace over socks. Loosen laces and pull the support up over the foot with the heel in the opening in back. Tighten the laces from bottom to top. The support should fit snug but not so tight to restrict circulation. Retighten for an even and comfortable fit. Wear inside a comfortable or athletic-type shoe.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L1902



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com