

Application Instructions

Product: Clavicle Support

Model: 260X

- Our most popular hook and loop closure clavicle support.
- Comfortable wide padding for proper immobilization.
- Felt back pad helps prevent irritation.
- Hook and loop closure for easy application.



Order #	Size	Description
2602	X-Small	Pediatric
2603	Small	Child/Youth
2605	Medium	Youth/Small Adult
2607	Large	Adult
2608	X-Large	Large Adult

HOW TO APPLY:

Will need assistance when applying and removing. Position the felt pad on the back between the shoulder blades. Wrap figure-8 arm straps around shoulders. Make certain the straps are not twisted and the arm pads are between the straps and the shoulders. Feed the straps through the plastic loops and pull tight as desired. Press the hook and loop closures to fasten. The arm pads should be adjusted so that they fit under the arms and in front of the shoulders. The straps should be tight enough to pull the shoulders back. Adjust as needed.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3660



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com