

Application Instructions

Product: Deluxe Carpal Tunnel Wrist Brace

Model: L-13X, R-12X

- Ideal for carpal tunnel syndrome, wrist sprains, strains or after cast removal.
- Constructed of durable, breathable canvas material for maximum wear and comfort.
- Anatomically cut to immobilize the wrist while permitting full finger movement.
- Hook and loop closure for easy adjustment.
- 8" length.



Order # Left	Order # Right	Size	Wrist Circum.
L - 131	R - 121	X-Small	4½" - 5½"
L - 133	R - 123	Small	5½" - 6½"
L - 135	R - 125	Medium	6½" - 7½"
L - 137	R - 127	Large	7½" - 8½"
L - 138	R - 128	X-Large	Over 8½"

SIZING:

Measure around center of wrist. Specify Right or Left.

HOW TO APPLY:

Unfasten closures. Pull brace over hand ensuring wide stay is on palm side of hand and plastic loop buckles on the top of the hand. Using free hand, thread the straps thru the plastic loops, pull until snug, and attach hook and loop. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3908



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com