

Application Instructions

Product: Deluxe Hinged Knee Brace

Model: 90X

- Medial and lateral hinge stays provide excellent support.
- Enclosed tubular donut buttress.
- Neoprene coated with 4-way stretch nylon fabric.
- Piping along thigh and calf areas of support to stop flexing.
- Tapered in back for comfortable fit.
- Hyperextension stops at 170°.
- Easy removable hinges for custom fitting.



Order #	Size	Knee Circumference
903	Small	12½" - 14½"
905	Medium	14" - 16"
907	Large	15½" - 18"
908	X-Large	17½" - 20"
909	2X-Large	19½" - 22"
910	3X-Large	21" - 24"

SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY: Unfasten all straps and closures. Insert foot into knee support and slide up over the knee. Align the medial/lateral hinge axis with the center of the knee joint. Support should fit snug but not so tight as to restrict circulation. Hinges should be aligned with the prominent bones on each side of the knee. Secure straps beginning with the top strap. Feed straps through rings and secure.

CARE INSTRUCTIONS: Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

WARNING: Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

SUGG HCPCS CODE: L1810 (PDAC Reviewed)



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com

©2010 Advanced Orthopaedics, Inc.