

# Application Instructions

**Product: Deluxe Universal Length Arm Sling**  
**Model: 2200**



- Envelope style.
- Can be folded to proper length to accommodate most patients.
- Soft foam neck pad for greater patient comfort.
- Slide buckle on strap for easy adjustment.
- Hook and loop closure.

Order #	Size
2200	Universal

## HOW TO APPLY:

Unfasten hook and loop closure on strap. Insert arm into the pouch with elbow in the pouch corner. If needed, fold-in the sling to shorten the length of the pouch and secure the hook tab on the loop closure. Bring strap across back and over the opposite shoulder. Run strap through plastic round loop; adjust to desired length with slide buckle adjustment. Press hook and loop closure against strap to secure. Adjust as needed.

## CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

## CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE: A4565**



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)