

Application Instructions

Product: Elastic Slip-On Ankle Support
Model: 270X

- Durable lightweight knitted elastic provides support with flexibility.
- Unique seamless construction will not irritate skin.
- Open heel and toe provides easy slip on.
- Fits either left or right ankle.
- Easy slip-on design.



Order #	Size	Ankle Circumference
2703	Small	7" - 8"
2705	Medium	8" - 9"
2707	Large	9" - 10"
2708	X-Large	10" - 11"

SIZING: Measure around the center of the ankle just above the ankle bone.

HOW TO APPLY:

Pull the support up over the foot with the heel in the opening in back. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A9270



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com