

# Application Instructions

## Product: Elastic Slip-On Wrist Support Model: 190X

- Durable knitted elastic provides support with flexibility
- Easy slip-on design
- Seamless construction will not irritate skin
- Fits either left or right wrist



Order #	Size	Wrist Circum.
1903	Small	4" - 6"
1905	Medium	6" - 8"
1907	Large	8" - 10"
1908	X-Large	10" - 12"

### SIZING:

Measure around center of wrist. Fits right or left.

### HOW TO APPLY:

Pull over hand and wrist like a glove with the thumb through the thumb hole. The support should fit snug but not too tight as to be uncomfortable or restrict circulation.

### CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

### CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** A4466



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)