

Application Instructions

Product: Elastic Wrist Brace

Model: L-14X, R-11X

- Ultra lightweight style wrist brace.
- Removable, adjustable palmar stay for maximum support.
- Hook and loop closure for easy adjustment.
- 7" length.



Order # Left	Order # Right	Size	Wrist Circum.
L - 141	R - 111	X-Small	4½" - 5½"
L - 143	R - 113	Small	5½" - 6½"
L - 145	R - 115	Medium	6½" - 7½"
L - 147	R - 117	Large	7½" - 8½"
L - 148	R - 118	X-Large	Over 8½"

SIZING:

Measure around center of wrist. Specify Right or Left.

HOW TO APPLY:

Unfasten closures. Pull brace over hand ensuring wide stay is on palm side of hand. Using free hand, pull the strap closures until snug, and attach hook and loop. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3908



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com