

# Application Instructions

**Product: Elastic Wrist Guard Support**  
**Model: 2900**

- Unique 3" wide elastic material adheres to hook closure strap for exact fit and easy adjustment
- Universal design fits a range of sizes
- Fits either left or right wrist



<b>Order #</b>	<b>Size</b>
2900	Universal

## **SIZING:**

Universal design fits a range of sizes. Fits right or left.

## **HOW TO APPLY:**

Wrap around the wrist until snug. The support should fit snug but not too tight as to be uncomfortable or restrict circulation. Attach the hook closure to secure in place.

## **CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

## **CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE: A4466**



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)