

# Application Instructions

## Product: K.S. Lace Wrist Brace

### Model: L-35X, R-34X

- Deluxe ultra-suede perforated material with flannel lining for added patient comfort.
- Recommended when wrist stabilization is needed.
- Two metal dorsal stays on the back of the brace helps prevent extension.
- Unique stockinette liner provides comfort and easy slip-on application.
- 7" length



Order # Left	Order # Right	Size	Wrist Circumference
L - 351	R - 341	X-Small	4½" - 5½"
L - 353	R - 343	Small	5½" - 6½"
L - 355	R - 345	Medium	6½" - 7½"
L - 357	R - 347	Large	7½" - 8½"
L - 358	R - 348	X-Large	Over 8½"

#### SIZING:

Measure around center of wrist. Specify Right or Left.

#### HOW TO APPLY:

Unfasten closures. Pull brace over hand ensuring wide stay is on palm side of hand and laces are on the inner side of the wrist. Using free hand, use the pull tab to tighten the lace closure until snug, and attach. Close the top strap around the thumb. Readjust if necessary.

#### CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

#### CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** L3908 (PDAC Reviewed)



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)