

Application Instructions

Product: K.S. Thumb Spica Support

Model: L-19X, R-19X

- Supports and limits the motion of the MP and CMC joints.
- Offers gentle thumb support.
- Indicated for deQuervain's syndrome and rheumatoid arthritis.



Order # Left	Order # Right	Size
L - 194	R - 190	Small/Medium
L - 196	R - 192	Medium/Large

SIZING:

Small/Medium is universal size for most women and Medium/Large is universal size for most men. Specify right or left thumb.

HOW TO APPLY:

Unfasten closures. Align the splint so the metal stay is parallel to the outside of the thumb (medial/inner side of the wrist). The metal stay can be bent to shape for a customized fit. The spica portion should fit around the base of the thumb. Using free hand, close the top strap around the thumb and wrap the straps around the wrist and hand starting at the bottom. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash, do not submerge in water but simply wipe clean with a damp cloth in cold water and mild soap. Use towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3999



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com