

Application Instructions

Product: Lightweight Spinal Orthosis Models: 1200-X

- Helps reduce lordosis through general abdominal compression while providing AP & ML stability.
- Chair-back orthosis lined with a 3/8" loop and compatible plush soft-foam.
- Back height 12", front 9 1/2".



| Order # | Size | Waist Measurement |
|---------|---------|-------------------|
| 1200-3 | Small | 30" - 35" |
| 1200-5 | Medium | 36" - 41" |
| 1200-7 | Large | 42" - 47" |
| 1200-8 | X-Large | 48" - 51" |

SIZING: Measure around waist at widest point.

HOW TO APPLY:

Apply in a standing position. Open the straps and center the brace in the middle of the back so the bottom is resting on the top of the buttocks. Wrap around the ends of the brace and secure the hook and loop closure in the front. Pull the overlapping elastic strap out and up until snug, secure the hook and loop strap closures over top of the front. Support should fit snug but not too tight. Readjust if necessary.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L0631



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com