

Application Instructions

Product: Lumbar Sacral Support with Insert

Pocket

Model: 50X-P

- All-elastic with double pull side panels for additional support.
- Specially constructed pocket accommodates moldable insert.
- Support may be worn with or without an insert.
- 9" in back tapering to 6" in the front.
- Hook and loop closure.



Order #	Size	Waist Measurement
503-P	Small	24" - 30"
505-P	Medium	30" - 34"
507-P	Large	34" - 38"
508-P	X-Large	38" - 42"
509-P	2X-Large	42" - 46"
510-P	3X-Large	46" - 50"
511-P	4X-Large	50" - 54"
512-P	5X-Large	54" - 58"

SIZING: Measure around waist at widest point.

HOW TO APPLY: Apply in a standing position. Unfasten all closures. Place the back of the support around back and upper buttocks area. Pull the sides of the support around until it holds the upper buttocks snugly, secure the hook and loop closure on the support base. Pull the overlapping abdominal strap until snug and attach hook and loop closures. Support should fit snug but not bind the groin area when sitting. Readjust if necessary. Note – Moldable Insert sold separately (Model# 550 8" width).

CARE INSTRUCTIONS: Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L0626 (when used with moldable insert)



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com

©2010 Advanced Orthopaedics, Inc.