

Application Instructions

Product: Lumbar Sacral Support with Removable Stays

Model: 59X

- Single overlapping belt provides additional abdominal support.
- All elastic construction comfortably enfolds body contour.
- Four shaped steel stays.



Order #	Size	Waist Measurement
593	Small	24" - 30"
595	Medium	30" - 34"
597	Large	34" - 38"
598	X-Large	38" - 42"
599-2	2X-Large	42" - 46"
599-3	3X-Large	46" - 50"
599-4	4X-Large	50" - 54"
599-5	5X-Large	54" - 58"
599-6	6X-Large	58" - 62"

SIZING: Measure around waist at widest point.

HOW TO APPLY: Apply in a standing position. Unfasten all closures. Place the back of the support around back and upper buttocks area. Pull the sides of the support around until it holds the upper buttocks snugly, secure the hook and loop closure on the support base. Pull the overlapping abdominal strap until snug and attach hook and loop closures. Support should fit snug but not bind the groin area when sitting. Readjust if necessary. Removable steel stays can be bent by a fitter for a customized fit.

CARE INSTRUCTIONS: Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com