

# Application Instructions

## Product: Lumbo Lite Lumbar Orthosis

### Models: 41X-B

- Streamline design allows for exact fit and comfort.
- Posterior and anterior padded panels (rigid and moldable) which provide lower back and abdominal compression.
- Extremely lightweight and form fitting.



Order #	Size	Waist Measurement
413-B	Small	26" - 32"
415-B	Medium	30" - 37"
417-B	Large	34" - 41"
408-B	X-Large	38" - 47"
419-B	2X-Large	44" - 54"
420-B	3X-Large	50" - 62"

**SIZING:** Measure around waist at widest point.

**HOW TO APPLY:** Apply in a standing position. Open the straps and center the posterior panel of the brace in the middle of the back so the bottom is resting on the top of the buttocks. Attached the anterior panel to one side of the brace and position centered low on the abdomen. Wrap around the other end of the support and secure the hook and loop closure on the anterior panel. Pull the overlapping elastic straps out and up until snug, secure the hook and loop strap closures over top of the front. Support should fit snug but not too tight. Readjust if necessary.

**CARE INSTRUCTIONS:** Remove the rigid posterior panel and fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:** Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** L0627



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)