

Application Instructions

Product: Lycra-Gel Ankle Brace

Model: 441

- Ultra lightweight functional ankle brace.
- Memory foam interface padding provides maximum comfort.
- Plastic shells provide medial/lateral stabilization to the ankle joint.
- Helps support and protect unstable or injured ankles.
- Allows normal dorsal and plantar flexion while controlling inversion and eversion.



Order #	Size	Brace Height
441	Regular	10 ½"
441C	Child	8 ½"

SIZING:

Universal design fits a most ankles. Fits right or left ankle. Adjustable heel width strap.

HOW TO APPLY:

Wear brace over socks. Place the plastic shells on the sides of the ankle with the foot on the bottom heel piece. Adjust the heel width if needed for a proper fit. While holding the plastic shells together, wrap the lower then the upper strap and fasten with hook and loop closures. The shells should be parallel with the leg and the brace centered on the ankle bone. Once both straps are fastened, readjust the straps to ensure a proper fit. Fit should be snug but not so tight it pinches or it constricts circulation. Wear over socks and with an athletic type-shoe.

CARE INSTRUCTIONS:

Spot clean with a damp cloth. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L4350



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com