

Application Instructions

Product: Neoprene Thigh Sleeve Support Model: 300

- Neoprene coated with 4-way stretch nylon fabric.
- Designed for relief and protection against injured quadriceps and hamstring muscles.
- Provides warmth and equal compression.



| Order # | Size | Thigh Circumference |
|---------|---------|---------------------|
| 300 | Small | 18" - 20" |
| 300 | Medium | 20" - 22" |
| 300 | Large | 22" - 24" |
| 300 | X-Large | 24" - 26" |

SIZING: Measure around the center of the thigh.

HOW TO APPLY:

Pull the support up over the knee and position on the center of the thigh with the seam in back. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

WARNING:

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

SUGG HCPCS CODE: A9270



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com