

Application Instructions

Product: Premium Arm Sling

Model: 221X

- Most popular arm sling.
- Thumb loop allows patient to relax hand.
- Slide buckle on strap for easy adjustment.
- Hook and loop closure.



Order #	Size
2211	X-Small
2213	Small
2215	Medium
2217	Large
2218	X-Large

HOW TO APPLY:

Unfasten hook and loop closure on strap. Insert arm into the pouch with elbow in the pouch corner. Bring strap across back and over the opposite shoulder. Run strap through plastic round loop; adjust to desired length with slide buckle adjustment. Press hook and loop closure against strap to secure. Adjust as needed.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4565



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com