

Application Instructions

Product: Premium Sized Knee Immobilizer

Model: 700X

- Lightweight, cool and comfortable.
- Provides support and stabilization for the leg and knee.
- Three posterior contoured stays for extra stability.
- Rigid medial and lateral stays for greater immobilization.
- Pro-pad insert included.



Order #	Size	Thigh Circumference
7003	Small	15" - 17"
7005	Medium	17" - 19"
7007	Large	19" - 21"
7008	X-Large	21" - 23"
7009	2X-Large	23" - 25"

SIZING: Measure around thigh 4" above the patella.

HOW TO APPLY:

Place the immobilizer behind knee using the contoured posterior stays as a guide for proper positioning. The inverted portion of the stays should be placed in the popliteal area (directly behind the knee). The medial and lateral stays should be aligned with the side of the leg. Wrap the sides of the immobilizer around and close in front. Secure straps immediately above and below the patella first. Work from the bottom upwards, closing the additional straps. Excess strap length may be trimmed to fit. Adjust if necessary.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L1830



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com