

Application Instructions

Product: Premium Tennis Elbow Support
Model: 2300

- Constructed of 2" web strap.
- Hook and loop closure for easy application.
- Fits either left or right forearm.



Order #	Size
2300	Universal

HOW TO APPLY:

Open the strap and slide over the forearm so it is positioned just below the elbow. Close the tension strap so it fits comfortably without restricting circulation. Adjust as needed.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A9270



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com