

Application Instructions

Product: Rib Belt Universal

Models: 2400, 2410

- Constructed of 6" wide breathable elastic material.
- Helps stabilize rib and sternum fractures by limiting chest expansion through compression.
- Female belt has contours around bust line and male belt is straight.



Order #	Size	Rib Measurement
2400	Universal Male	24" - 60"
2410	Universal Female	24" - 60"

SIZING: Measure around center of center just below bust.

HOW TO APPLY:

Apply in a standing or laying position. Open belt to its fullest length with the care tag on the inside. Position the rib belt around the rib cage. The hook should be on the right. Stretch both sides and wrap the sides without the hook around the body first. Secure the hook onto the loop part of the support where needed for desired compression. Support should fit snug but not too tight. Readjust if necessary.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com