

# Application Instructions

## Product: Sacral Support with Removable Pad Model: 52X

- Elastic construction with four stays for added support.
- Pad in sacral area provides additional support.
- 9" in back tapering to 4" in the front.
- Hook and loop closure.



Order #	Size	Waist Measurement
523	Small	24" - 30"
525	Medium	30" - 34"
527	Large	34" - 38"
528	X-Large	38" - 42"
529	2X-Large	42" - 46"

**SIZING:** Measure around waist at widest point.

### HOW TO APPLY:

Apply in a standing position. Unfasten all closures. Place the back of the support around back and upper buttocks area. The removable pad should be centered on the lower back just above the buttocks in the natural curve of the back. Pull the sides of the support around until it holds the upper buttocks snugly, secure the hook and loop closure on the support base. Support should fit snug but not bind the groin area when sitting. Readjust if necessary.

### CARE INSTRUCTIONS:

Fasten closures, remove pad. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

### CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** A4466



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)