

Application Instructions

Product: Silicone Elastic Ankle Support
Model: 45X

- Viscoelastic silicone insert helps provide relief from pain, swelling and inflammation.
- Three dimensional knitted elastic.
- Open heel and toe provides easy slip on.
- Easy slip-on design.
- 7" Sleeve.



Order #	Size	Ankle Circumference
453	Small	7 7/8" – 8 3/8"
455	Medium	8 3/8" – 9 1/4"
457	Large	9 1/4" - 10"
458	X-Large	10" – 10 3/4"

SIZING: Measure around the center of the ankle just above the ankle bone.

HOW TO APPLY:

Pull the support up over the foot with the heel in the opening in back. The J-shaped viscoelastic insert should fit around the ankle bone. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A9270



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com