

# Application Instructions

**Product: Sport Knee Sleeve Support**

**Model: 30X**

- Neoprene coated with 4-way stretch nylon fabric.
- Tapered in back for comfortable fit.
- Open patella relieves pressure on the patella area.



Order #	Size	Knee Circumference
303	Small	12½" - 14½"
305	Medium	14" - 16"
307	Large	15½" - 18"
308	X-Large	17½" - 20"
309	2X-Large	19½" - 22"
309-9	3X-Large	21" - 24"

**SIZING:** Measure around the center of the knee with leg extended.

## HOW TO APPLY:

Pull the brace over the knee with the opening centered on the kneecap. The support should fit snug but not so tight to restrict circulation.

## CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

## CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

## WARNING:

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

**SUGG HCPCS CODE:** A4466



326 Main Street  
Harleysville, PA 19438

[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)