

Application Instructions

Product: Tri-Panel Knee Immobilizer-Universal Model: 70X

- Tri-Panel design for more anatomical fit.
- Movable side panels for medial and lateral stay positioning.
- Three contoured posterior stays.
- Two medial and two lateral stays.
- Hook and loop closures.



Order #	Size	Length
703	Universal	14"
705	Universal	20"
707	Universal	24"

SIZING: Universal design fits a range of sizes, size based on height/length.

HOW TO APPLY: The tri-panel style has adjustable side panels to fit a wide range of sizes and can be angled for a tapered fit. Determine the appropriate width by removing the panels and placing in the desired position. Angling the panels will improve the fit for wider thighs or thinner calves. The medial and lateral stays should still be aligned with the sides of the leg. Fully attach the hook on the side panels to the base. Place immobilizer behind knee using the contoured posterior stays as a guide for proper positioning. The inverted portion of the stays should be placed in the popliteal area (directly behind the knee). Close the panels of the immobilizer. With the immobilizer in correct position, start securing straps immediately above and below the patella first. Work from the bottom upwards, closing the additional straps. Use the removable piece of double sided hook for a more customized fit. Excess strap length may be trimmed to fit. Re-adjust if necessary.

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L1830



326 Main Street
Harleysville, PA 19438

www.advanced-orthopaedics.com

©2010 Advanced Orthopaedics, Inc.