

Application Instructions

Product: Universal Wrist/Forearm Brace

Model: L-160, R-150

- Removable and adjustable palmar stay for maximum support and immobilization.
- Brace can be easily removed and re-applied once properly fitted.
- Hook and loop closure for easy adjustment.
- Universal sizing, adjustable to fit a range of sizes.
- 11" wrist/forearm length.



Order # Left	Order # Right	Size
L - 160	R – 150	Universal

SIZING:

Universal design fits a range of sizes. Specify Right or Left.

HOW TO APPLY:

Unfasten closures. Pull brace over hand ensuring wide stay is on palm side of hand and plastic loop buckles on the top of the hand. Using free hand, thread the straps thru the plastic loops, pull until snug, and attach hook and loop. Close the top strap around the thumb. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3908



326 Main Street
Harleysville, PA 19438
www.advanced-orthopaedics.com