

# Application Instructions

**Product: Velveau Immobilizer**  
**Model: 220X**



- Breathable, cool cotton material.
- Webbing swathe strap for shoulder immobilization.
- Soft foam neck pad for greater patient comfort.
- Slide buckle on strap for easy adjustment.
- Thumb loop.
- Hook and loop closure.

Order #	Size	Strap Length
2203	Small	36"
2205	Medium	39"
2207	Large	43"

## HOW TO APPLY:

Unfasten hook and loop closures on straps. Insert arm into pouch with elbow in the pouch corner. Bring strap across back and over the opposite shoulder. Run strap through plastic round loop; adjust to desired length with slide buckle adjustment. Press hook and loop closure against the strap to secure. Soft foam pad should be positioned at neck and shoulder for greater comfort. To help secure the shoulder, bring the bottom strap around the waist, adjust length with slide buckle, and secure the closure. Adjust as needed.

## CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

## CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE: A4565**



326 Main Street  
Harleysville, PA 19438  
[www.advanced-orthopaedics.com](http://www.advanced-orthopaedics.com)